25/12619771/0418

## Information for use

### Composition

1000 ml solution contain:  
Water for Injection 1000 ml

### Indications

For irrigation and cleansing during surgical intervention  
For irrigation of wounds and burns  
For moistening wound tamponades, cloths and dressings  
For rinsing and cleansing of bladder catheters  
For irrigation and cleansing in stoma care  
For rinsing and cleansing of instruments and equipment  
For filling respiratory air humidifiers (the use of container equal or higher than 100 ml is recommended)  
For mechanical irrigation of the eyes (the use of 30 ml container is recommended)

### Dosage

The amount of fluid depends on the clinical condition.

### Overdosage

Overdosage during wound irrigation and intra-operative application can lead to absorption of irrigation fluid, that can lead to hypervolaemia in the form of hypotonic hyperhydration (hyponatraemia).  
Headache, nausea, restlessness and disorientation are symptoms of hyperhydration. In severe cases coma-like states can occur.

Please do not make any decisions of medical relevance before consulting your health care provider

CE 0123

# Sterile Water for Irrigation

sterile and endotoxin-free

### Countermeasures in cases of overdosage

Discontinuation of irrigation, correction of the water and electrolyte balance depending on the particular clinical situation.

### Warnings and precautions

Do not use for infusion.  
Only use the solution if the seal is undamaged and the solution is clear.  
Opened containers must not be saved, quantities left over must be discarded.  
Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.  
Store out of reach of children.  
Aqua B. Braun must not be used for intrathecal irrigation procedures.

### Adverse side effects

None are known.

### Expiry date

The solution must not be used after the expiry date given on the label.

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B. Braun Melsungen AG