



ASKINA® HEEL

HYDROCELLULAR HEEL DRESSING B. Braun Wound Care. Right. From the start.

Askina® Heel

Anatomically shaped hydrocellular heel dressing

Ready to use concept

No fixation device needed

Innovative anatomical design covering the malleolus

Pressure relieving foam, helps prevent stage I ulcers

Good protection against friction and shear stresses

Askina® Heel is an anatomically shaped, two layered, non adherent foam dressing, which includes a securement foam strap and two self adhesive hooks that allow dressing to remain in place even during movement.









How to apply Askina® Heel

- ① Apply self adhesive labels on each side of Askina® Heel
- ② Apply gently the dressing on the heel area and fix with the strap
- 3 Correctly positioned Askina® Heel: the ankle is covered
- 4 The dressing can be also fixed with a compression stocking

Indications

Management of moderately to heavily exuding, partial to full thickness wounds on the heel

- ► Stage I-IV pressure ulcers
- Surgical incisions
- ▶ 1st and 2nd degree burns



- The strap can be opened and closed many times to ensure that the dressing is securely and comfortably held in place
- ► Wound can be easily inspected, without patient discomfort



	Size	Pcs/Pack	Article number
Askina® Heel	surface = 225 cm ²	5	7240105

Manufactured by:

B. Braun Hospicare Lt.| Collooney | Co. Sligo | IRELAND www.bbraun.com/wound-management

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and/or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non-binding documents and photographs. For healthcare professional use only.